


Start here.



- 1 Download the Bose Hear app.**
Visit app.Bose.com/SC using your phone's web browser. Or, download from the App Store or Google Play store.

The app lets you control your hearing aids and teaches you how to wear them.

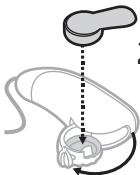

- 2 Open the battery doors and insert the batteries in them** with the stickers facing up and the rounded sides facing down.

Then, peel the stickers off and fully close the doors. You feel two clicks.



- 3 Follow the app instructions.**

Need help? See reverse for more information.



We're here to help.



For more detailed instructions, refer to the “Set up the hearing aids” section in the owner’s guide.



Visit the Bose product support website for videos, downloads, articles, and other resources.

worldwide.bose.com/Support/SoundControl



862367-0010